
Aliaxin[®]SV

SUPERIOR VOLUME

Cross-Linked
Hyaluronic Acid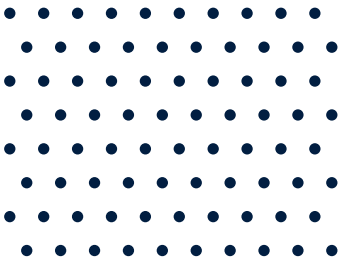

Hydrolift[®]
ACTION

hydroACTION ●○○○○○
liftACTION ●●●●●●
CROSS-LINKING ■■■■■■


1


2


3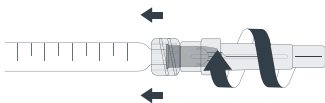

4

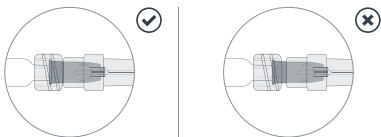5

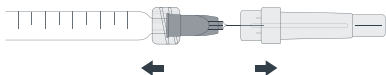IT

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EN

ASSEMBLING THE NEEDLE ON THE SYRINGE

For optimal handling of Aliaxin® SV Superior Volume products, it is important to assemble the needle on the syringe in accordance with the 5 steps shown in the diagram (see figures 1 to 5).

1

Firmly hold the glass syringe and the luer-lock between the thumb and the index fingers. Grasp the protective cap and unscrew it.

2

Remove needle cap, then firmly insert the needle thread on the end of the syringe.

3

Gently screw the needle in a clockwise direction. Continue to screw until the thread comes into contact with the body of the syringe.

4

Check that the needle is correctly fitted onto the thread. Failure to follow these precautions could lead to risks of the needle dropping out and/or leaks from the luer-lock.

5

With the syringe held in one hand and the cap in the other, remove the needle protective device by pulling firmly.

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COMPOSITION

Hyaluronic acid * 25 mg / mL

*In the form of cross-linked sodium hyaluronate

Physiological pH phosphate buffer pH QS

DESCRIPTION

Aliaxin® SV Superior Volume is a viscoelastic hyaluronic acid gel, sterile, apyrogenic and of non-animal origin.

INDICATIONS

Aliaxin® SV Superior Volume is indicated for:

- volume restoration and the filling of cutaneous depressions and wrinkles
- facial reconstruction of volumes in the event of light-to-moderate lipodystrophies, i.e. due to HIV treatment

Aliaxin® SV Superior Volume is indicated for the augmentation of cheeks, cheekbones and facial contours, as well as the filling of very deep wrinkles.

STERILISATION

The content of the Aliaxin® SV Superior Volume syringes is sterilised using moist heat. The needles are sterilised using radiation

INTENDED USE

Aliaxin® SV Superior Volume is a filling gel that is injected into the subcutaneous layer in order to restore volume in the required proportions.

CONTRAINDICATIONS

Aliaxin® SV Superior Volume must not be used:

- for injections other than subcutaneous ones, and more specifically it absolutely must not be injected into blood vessels. Do not inject intravascularly.
- in the periorbital and glabellar area.

Do not inject in cases of:

- Known hypersensitivity to any of the components,
- A history of severe allergies or anaphylactic shock.
- In combination with skin peeling treatment, laser treatment, ultrasound-based treatment or a technique based on an active dermal response.
- Inflammatory and/or infectious cutaneous problems on or near the area to be treated (acne, herpes, etc.) or tumours

Do not inject into patients:

- Suffering from autoimmune diseases,
- Suffering from epilepsy which is not controlled by treatment,
- That tend to develop hypertrophic scars
- Suffering from porphyria
- That are pregnant or breast-feeding
- That are under 18 years of age.

PRECAUTIONS FOR USE

Before starting treatment, patients should be questioned about their medical history, and be informed of the treatment's predictable results and its potential adverse effects.

It is not advisable to inject Aliaxin® SV Superior Volume products into patients that have taken vitamin E or vitamin C in large doses, aspirin, anti-inflammatories or anti-coagulants.

This device is intended for single use only. Sterility is not guaranteed in the event of re-use.

Do not use the product if the box is damaged or if the expiration date has passed. It is not advisable to inject into an area which has already been treated with a semi-permanent or permanent implant.

POST-TREATMENT CARE

After treatment, patients must be advised:

- Not to apply make-up in the 12 hours following the injection
- Not to expose themselves in the two weeks following the treatment to extreme temperatures (intense cold, sauna, hammam, etc.) or sunlight, nor to take sunbed sessions in the week following the treatment.

Do not apply pressure to the treated area.

DOSAGE

Aliaxin® SV Superior Volume may only be injected by practitioners who are trained in the techniques of injecting filling products and who are authorised to perform injections.

Practitioners must inject the amount depending on the desired effect, but without over-correcting: they must carry out at the most 100% of the desired correction, without going further.

Aliaxin® SV Superior Volume is a biodegradable gel that is absorbed over time and may require several injections before a satisfactory result is obtained.

The practitioner may suggest subsequent sessions to maintain the desired effect. The frequency of these sessions is determined by the results of previous injections.

The average duration depends on several factors: the type of patient's skin, the severity of the wrinkle to be corrected, type of injection (superficial, medium or deep dermis) and the injected volume.

MODE OF ADMINISTRATION

Before injection:

- Disinfect the treatment area.
- Remove the air by gently pushing down on the plunger to make a small amount of the product appear at the tip of the needle.
- Do not mix with other products before injection.

Do not bring together with quaternary ammonium salts due to a risk of precipitate. In particular, do not bring together with equipment sterilised with benzalkonium chloride.

Injection:

- It is recommended to perform the injection using the 27G needles provided in the box.
- Aliaxin® SV Superior Volume is injected evenly on the areas to be treated into the subcutaneous layer
- An injection which is too superficial or one whose injected volume is excessive may cause bumps on the treated area.

After injection:

- After the injection, it is recommended to massage the treated area to enable the product to spread evenly.
- During the injection, if a sensation of obstruction or pressure is felt, the injection must be stopped and the needle must be changed.
- Always dispose of the syringe, needle and unused product immediately after the treatment and do not use them again. This is to prevent any contamination of the remaining product and to avoid the associated risks, especially risks of infection.

These items must be disposed of in accordance with applicable medical practice as well as with institutional or local and national rules.

SIDE EFFECTS

The patient must be informed that:

- symptoms of inflammation on the implant site appear soon after the injection or with a delayed onset.
- in general, the injection of a medical device carries the risk of infection.

Reactions that may occur due to the injection are as follows:

- Oedema
- Haematoma,
- Erythema,
- Itchiness,
- Swelling,
- Pain or tenderness at the implant site,
- Induration or nodules at the injection point, perhaps leading to granuloma.
- Cutaneous necrosis
- Immediate hypersensitivity, perhaps leading to anaphylactic shock
- Dyschromia at the injection site

If inflammatory reactions persist beyond one week or if any other side effect occurs, the patient should inform the practitioner as soon as possible. The practitioner should then carry out the appropriate treatment.

Adverse effects should be reported to manufacturer and/or local distributor of the product.

STORAGE CONDITION

The expiration date is indicated on the packaging.

Store between 2°C and 25°C, away from light. Do not freeze.

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Rose Pharma S.A.
IBSA Farmaceutici Italia Srl